90.1122/6 (April 2021)

en

REF TR204

**DESCRIPTION OF DEVICE & CONTENTS****Trio Elisse®** Sting free skin barrier spray (50ml)**INTENDED USE:**

Protection of skin from the damage associated with body waste, enzymes and adhesives.

**INSTRUCTION FOR USE:**

- Clean and dry the skin.
- Hold approximately 10cm from the area to be treated.
- Apply a uniform coating in a sweeping motion.
- Ensure the area is completely dry before applying adhesive products such as appliances or dressings.
- Replace at every dressing or appliance change to maintain complete protection.

**INFORMATION:**

Removal is not normally necessary. However Trio Elisse® can be removed easily using Trio Elite® medical adhesive remover.

Sprays at any angle. Soothes reddened and sore skin. Reduces itching.

**CAUTION:**

- Avoid direct use on wound beds.
- If in doubt, patch test a small area prior to use.
- Should skin irritation occur, discontinue use and consult a healthcare professional.
- Avoid inhalation.
- For external use only.

**WARNINGS:**

Use only as directed. Pressurised container: may burst if heated. Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking. Do not pierce or burn even after use. Protect from sunlight. Do not expose to temperatures exceeding 50°C. Keep out of reach of children.

**EXTREMELY FLAMMABLE****SERIOUS INCIDENT REPORTING**

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

**SAFE DISPOSAL INSTRUCTIONS**

Do not dispose in the toilet. Dispose of product according to national or local laws

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**Trio Healthcare Ltd.**