90.1118/6 (April 2021)

en

REF TR201
DESCRIPTION OF DEVICE & CONTENTS**Trio Elite®** Sting free medical adhesive remover spray (50ml)**INTENDED USE:**

Removal of adhesives, dressings and appliances from skin.

INSTRUCTION FOR USE:

1. Spray around the edge of the adhesive and wait a few seconds.
2. Gently peel adhesive away from the skin using additional spray to ease removal if necessary.

INFORMATION:Dries in a few seconds.
Sprays at any angle.**CAUTION:**

- Avoid direct use on wound beds.
- If in doubt, patch test a small area prior to use.
- Should skin irritation occur, discontinue use and consult a healthcare professional.
- Avoid inhalation.
- For external use only.

WARNINGS:

Use only as directed. Pressurised container: may burst if heated. Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking. Do not pierce or burn even after use. Protect from sunlight. Do not expose to temperatures exceeding 50°C/122°F. Keep out of reach of children.

EXTREMELY FLAMMABLE**SERIOUS INCIDENT REPORTING**

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

SAFE DISPOSAL INSTRUCTIONS

Do not dispose in the toilet. Dispose of product according to national or local laws

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