en

**Description of device & contents**

Trio Silken® gel

**INTENDED USE:** As a filler / sealant and protective skin barrier for use on exposed skin around stoma, wound and fistula sites. A filler for skin folds and uneven skin surfaces.

**INSTRUCTIONS FOR USE:**

- 1 To open the tube. Unscrew the cap. Puncture the tube closure with the reverse side of the cap.
- 2 Ensure the skin is clean and dry.
- 3 Squeeze the silicone gel from the base of the tube, to fill depressions and skin contours. Do not spread.
- 4 Apply the ostomy wafer or pouching system as normal while the Trio Silken® gel is still wet.
- 5 Trio Silken® gel will set on the skin or underneath the appliance during wear time.

**REMOVAL:**

- 1 Peel the Trio Silken® gel from the skin surface. The gel may peel off with the appliance.

**IMPORTANT NOTE:**

Trio Silken® gel is not a glue.

**CAUTION:**

For external use only.  
Do not apply into an open wound or skin ulcer.  
Avoid contact with eyes.  
Should skin irritation occur, discontinue use and consult a healthcare professional.

**SERIOUS INCIDENT REPORTING**

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

**SAFE DISPOSAL INSTRUCTIONS**

Do not dispose in the toilet. Dispose of product according to national or local laws

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