en

**Description of device & contents**  
 Trio Siltac® Silicone ostomy seals  
 Trio Silvex® Silicone convex seals

Trio Siltac® Flat Ostomy Seal		
N°	REF Code	Size (mm)
1	TR1020 / TR1020 - 10	20-28
2	TR1028 / TR1028 - 10	28-35
3	TR1035 / TR1035 - 10	35-44

Trio Silvex® Convex Ostomy Seal		
N°	REF Code	Size (mm)
1	TR1020CX	20-30
2	TR1030CX	30-40

**INTENDED USE:** To provide a seal between the stoma, peri-stomal skin and the ostomy pouching system.

- Flat seal for use with protruding stoma.
- Convex seal for use where mild convexity is required.

**INSTRUCTIONS FOR USE** To ensure a secure seal is achieved, it is important to select the correct seal size to fit the stoma. Should leakage occur you may select the next size down to achieve a tighter seal.

#### APPLICATION

- Clean area with water and ensure that the skin is dry. Attention: The seal will not stick to wet skin.
- Remove by gently rubbing the edge of the seal toward the centre; then using thumb and forefinger carefully but firmly pull from the packet. The seal will normally be adhered to the packet.
- Stretch Trio Siltac® or Trio Silvex® over the stoma and allow it to retract. Once the seal has retracted around the stoma, secure to the body by gently pressing on the seal until secure and comfortable.
- Place your ostomy pouching system directly onto the surface of the seal and hold in place with gentle pressure until secure.

#### REMOVAL

- Gently peel the pouching system from the body.
- Remove the seal from the skin surface. The seal may peel off with your pouching system.

**Caution:** Should peri-stomal skin irritation occur, discontinue use and consult a healthcare professional.

**SERIOUS INCIDENT REPORTING:** Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

**SAFE DISPOSAL INSTRUCTIONS:** Do not dispose in the toilet. Dispose of product according to national or local laws

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